



## 5 REASONS TO BUY

**1** DentalVibe alleviates the anxiety of oral injections by applying a light vibration to the nerves and tissue around the injection site to help block pain signals from the brain.

**2** Featured as one of "the best products for 2020" in Gordon J. Christensen's *Clinicians Report*.

**3** Supported by clinical studies performed by the Tufts University School of Dental Medicine and the University of the Pacific Arthur A. Dugoni School of Dentistry.

**5** The new DentalVibe mobile application and locator program helps identify new patients who are seeking a pain-free experience and directs them to your practice.

**4** Patients who experience pain-free injections are more likely to refer their friends to your practice.



### DentalVibe®



Fear of pain from dental treatment, specifically injection pain, can be a major barrier for patients. By using a light vibration as a 'counterstimulus' to confuse the brain and block the pain of injection, the DentalVibe allows me to provide pain-free anesthesia and remove one of the barriers that prevents fearful patients from receiving the dental care that they need.

Jason Goodchild, DMD  
Jenkintown, Pennsylvania

dentalvibe.com

#### SPECIFICATIONS

- Illuminated tip for improved visibility in the oral cavity
- Induction charging base with power cord
- Starter pack of 25 Flexi Tips included

## dentalvibe

DentalVibe  
dentalvibe.com • 877-503-8423

# PAIN-FREE DENTISTRY Begins with a Pain-Free Injection

Millions fear the dentist because of painful oral injections. DentalVibe® alleviates the anxiety of oral injections by massaging the nerves and tissue around the injection site for better dental experiences, helping once petrified patients smile every day!

Drop the old-school and ineffective cheek shake and start giving real pain-free injections with DentalVibe!

#### CAN YOU PROVIDE PAIN FREE INJECTIONS?

##### FOLLOW THIS PROTOCOL:

- light touch
- 15 seconds of vibration
- injection within 3mm of the prong



*The DentalVibe is my go-to tool for delivering better local anesthetic injections, with my patients usually reporting*

*no injection discomfort at all! The DentalVibe is simple to use – after installing a new tip, I lightly place the vibrating prongs in the area of injection for approximately 15 seconds, and while still holding the DentalVibe on the tissue I aim my injection point as close to the prongs as possible."*

Jason Goodchild, DMD



www.dentalvibe.com  
877-503-8423

## dentalvibe